



# Lab help

PAGE 1:2

## Implant - Edentulous Case Checklist for Hybrid Success

We hope this guide will help and assist you.

If you have any questions or need assistance you can contact us at [800-336-3053](tel:800-336-3053) or visit

our website and click on the icon as shown to the right, which you can find along the bottom right hand corner of our website at [burbankdental.com](http://burbankdental.com). Just request Lab Help and we will be glad to assist you.

Also, check out our Resources section when visiting our website.



### 1 First Appointment

#### 1 Pre-Op Models

- 1 Impression for Replica of Refined Denture
  - a Do a reline impression on current denture.
  - b Take an impression of current denture, using denture duplicating flask. (Lang Dental)
  - c Bite Registration with relined denture(s)



### 2 Photos, Proposed Plan

#### 2 Photos

- 2 Photos
  - a Full face smile (forehead to chin)
  - b Lip at rest close-up
  - c Smile close-up
  - d Occlusal views



#### 3 Proposed Plan

- 3 Proposed Plan
  - a Number of implants
  - b Size of implants
  - c Location of implants
  - d VDO change or remain as current
  - e Type of restorations proposed
  - f Fixed screw retained
  - g Fixed cementable
  - h Hybrid, Locator Bar, or Hader Bar
  - i Direct attachment, or any other goals or concerns

### 3 Proposed Plan Continued

-  Lab Delivery: Bite Blocks or Scanning Guide
-  Lab Delivery: Bite Block and Verification Jig

#### 1a CT Scanning Appliance


If current denture is going to be the design template, Burbank Dental Lab can deliver CT Scanning Appliance or Surgical Guide.

 [Go to Page 2: #3 Lab Delivery](#)

#### 1b Wax Bite Blocks

If current denture is going to be substantially altered, then Burbank Dental Lab will deliver the following:

- a 2-sets of Bite Blocks with centrals setup
- b One to be used *edentulous to edentulous*, the other set is taken against the opposing dentures.

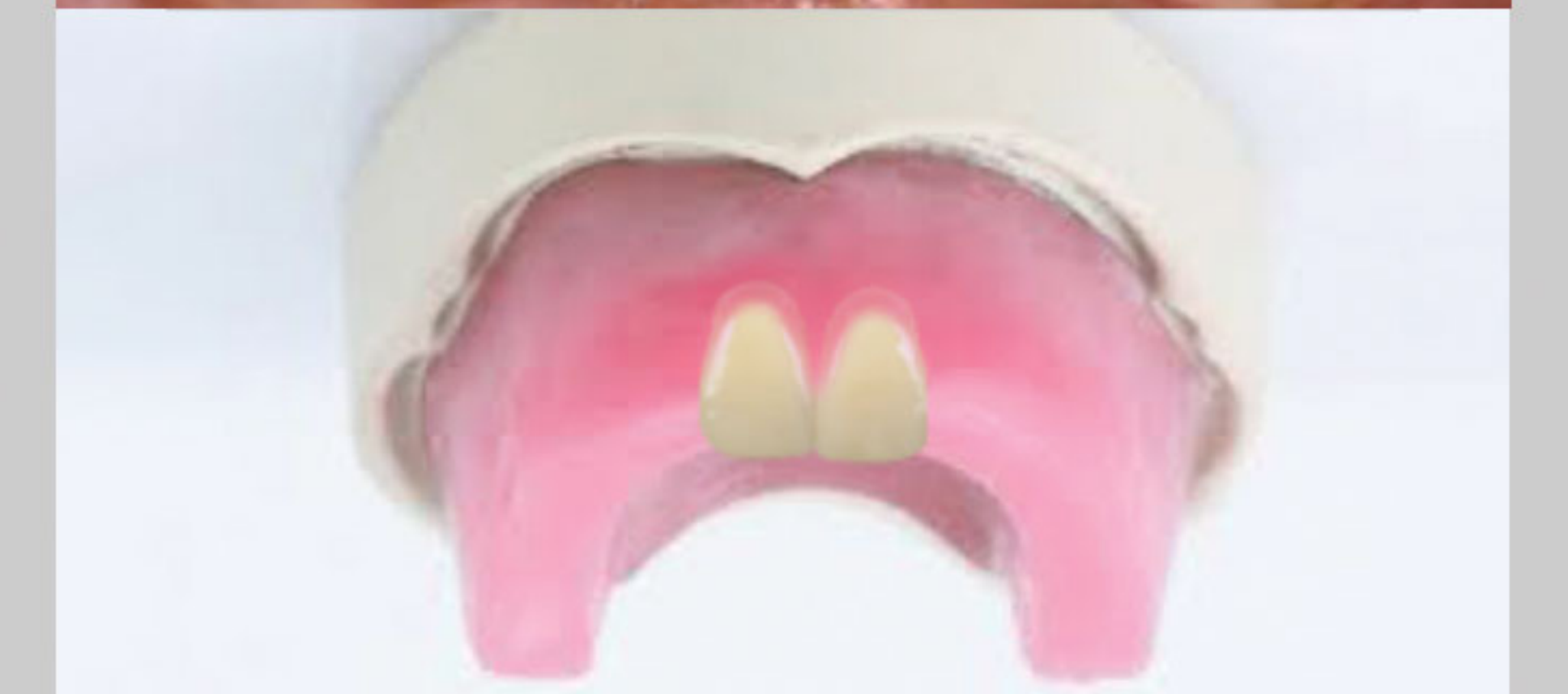



# Lab help

PAGE 2:2

## Implant - Edentulous Case Checklist for Hybrid Success

We hope this guide will help and assist you.

If you have any questions or need assistance you can contact us at **800-336-3053** or visit




our website and click on the icon as shown to the right, which you can find along the bottom right hand corner of our website at [burbankdental.com](http://burbankdental.com). Just request Lab Help and we will be glad to assist you.

Also, check out our Resources section when visiting our website.

### 1 Clinical Records & Steps

- 1 Bite Blocks
  - a Try-In Bite Blocks (verify or reset Centrals to Ideal)
  - b Measure Shimbashi from centrals CEJ to lower incisors CEJ

 Lab Delivery: Wax set-up

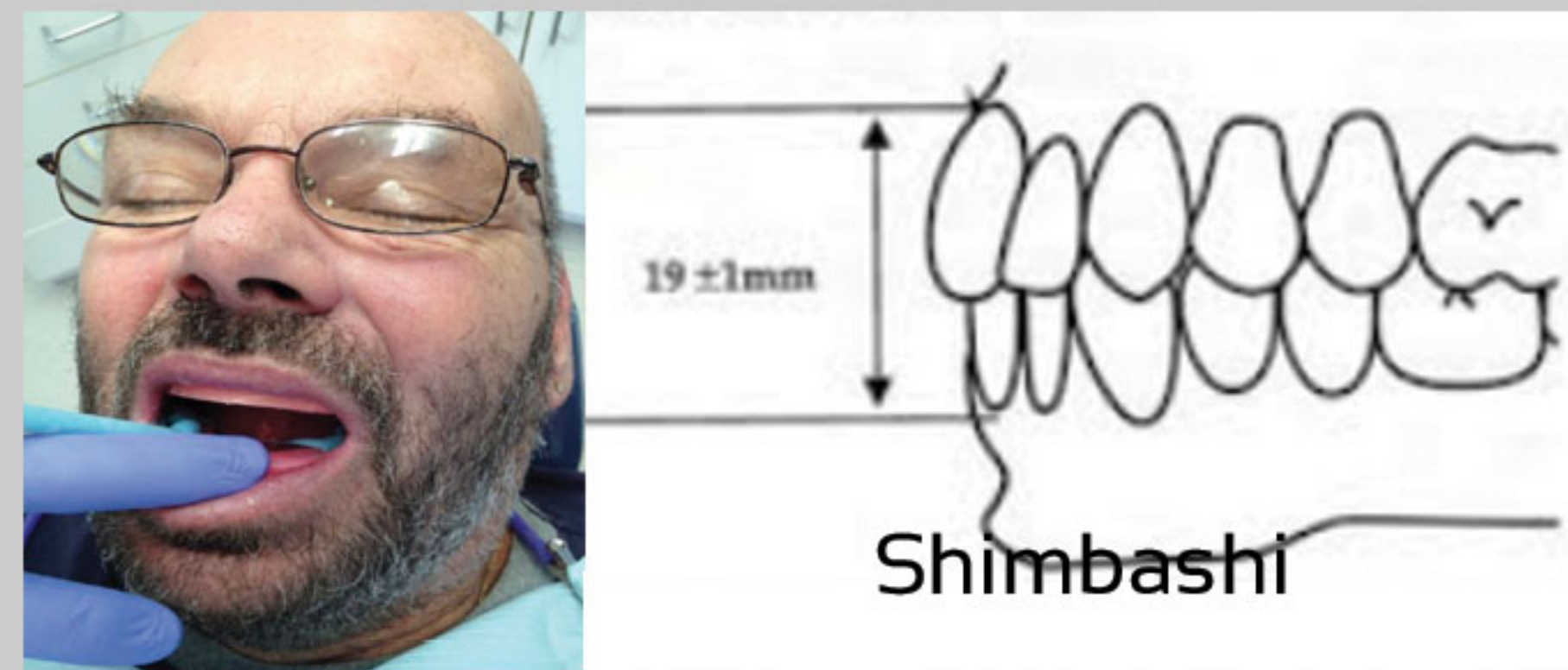

### 2 Trial Denture Try-In

- a Trial Denture try-in prior to bar, and could be prior to implant placement



 Lab Delivery: CT Scanning Guide or Surgical Stent

If old denture is not going to be used.

Replica of new set-up is used for CT Scanning Stent or Surgical

### 2 Clinical Records & Steps

- 2 After Implant Integration
  - 2 Implant Pick-Up Impression (open tray)
    - a \*Use denture duplicate as open impression tray. Request Handle & Holes from Burbank Dental Lab.

 Lab Delivery: Screw retained set-up and Verification Jig

- b Bite block over implant model (screw retained)
- c Verification Jig



### 3 Clinical Records & Steps

- 1 Clinical Records & Steps
  - a Try-In Verification Jig
  - b Try-in bite block use PVS bite registration over bite block.
  - c Photos of set-up (3 photos: Face/Smile, close-up smile, close-up lip at rest).



 Lab Delivery: Final Bar with set-up

### 2 Bar With Wax Set-Up

Clinical Steps

- a Verify wax set-up
- b Once set-up is verified final hybrid is processed.